

## Situation 14 Module Mismatch: TWELVE flashes of the status LED

Twelve flashes of the status LED occur when the data held by the RemG80 is incorrectly programmed for the power chair it is being used on.

Try having the RemG80 joystick reprogrammed for the specific model of power chair and the accessories that go with the chair.

Another problem could be compatibility between the Dynamic modules in the system. If this happens, the power chair will be disabled by the electronics. If this happens, assure that the programming is correct for the chair and the accessories on the chair.



**In this case, contact Pride Power Chair Technical Service at 1-877-800-1248.**

---

### Notes: